

MIMICS^{3D} One-Year Results

Summary

A prospective observational registry evaluating the BioMimics 3D Vascular Stent System in a real-world clinical population with a dedicated subgroup analysis of device performance as a complementary treatment in procedures involving drug-coated balloons. MIMICS-3D enrolled 507 patients across 23 clinical sites in Europe.

24% of subjects enrolled in MIMICS-3D had Critical Limb Ischemia

38% of lesions had moderate to severe calcification

| Baseline Patient Demographics | | N=507 Subjects |
|-------------------------------|---------------------|-----------------|
| Age | Mean years ± SD (N) | 70.1 ± 10 (507) |
| Gender | % Male | 65.5% (332/507) |
| Risk Factors | Diabetes Mellitus | 36.9% (187/507) |
| | Smoker Current | 37.7% (191/507) |
| Rutherford category | 0 | 0.4% (2/504) |
| | 1 | 1.2% (6/504) |
| | 2 | 17.1% (86/504) |
| | 3 | 57.3% (289/504) |
| | 4 | 7.5% (38/504) |
| | 5 | 14.3% (72/504) |
| Ankle Brachial Index | Mean ± SD (N) | 0.6 ± 0.3 (417) |

| Baseline Lesion Characteristics | | N=507 Subjects (518 lesions) |
|---------------------------------|-------------------|------------------------------|
| Reference Vessel Diameter (mm) | Mean ± SD (N) | 5.3 ± 0.7 |
| Lesion Location (%) | Mid to Distal SFA | 62.9% 326/518 |
| | Prox. Pop | 7.3% (38/518) |
| Diameter Stenosis (%) | Mean ± SD | 94.6 ± 8 (518) |
| Occlusions | Total | 56.8% (294/518) |
| Lesion Length (mm) | Mean ± SD | 127 ± 92.4 |
| | Grade 0 | 17.6% (91/518) |
| Calcification | Grade 1 | 29.3% (152/518) |
| | Grade 2 | 24.1% (125/518) |
| | Grade 3 | 14.7% (76/518) |
| | Grade 4 | 13.9% (72/518) |

Study Principal Investigator:

Michael Lichtenberg MD, Arnsberg, Germany

Enrolment complete:

N=507

Clinical Sites: 23 Pan European

Follow Up: 3 Years

Primary Endpoints:

Safety – Composite of major adverse events (MAE), death, major amputation performed or CDTLR through 30 days.

Effectiveness – Freedom from clinically-driven target lesion revascularisation through 12 months.

Results

Primary Safety Endpoint: 30 days

| | |
|--------------------------------|--------------|
| CDTLR* | 0.4% (2/490) |
| Major amputation target limb** | 0.4% (2/491) |
| Death*** | 0.4% (2/491) |
| Any MAE | 1.2% (6/492) |

Primary Effectiveness Endpoint:
1 Year
Freedom from CDTLR

89%

DCB vs non-DCB

Subgroup analysis of device performance as a complementary treatment in procedures involving drug-coated balloons.


| | BioMimics 3D with DCB | BioMimics 3D without DCB | Overall result |
|--------------------------|-----------------------|--------------------------|-----------------|
| Freedom from CDTLR @12mo | 89.5% | 88.5% | 89.0% [p > .88] |

*2 events adjudicated: occlusions Day 13, 18

**2 events adjudicated:

(1) Day 8, RCC 6 subject unplanned, above knee amputation; (2) Day 3, RCC 6 subject, unplanned, below knee amputation

***2 events adjudicated: (1) surgical complication following closure system failure; (2) MOF during palliation for ALL

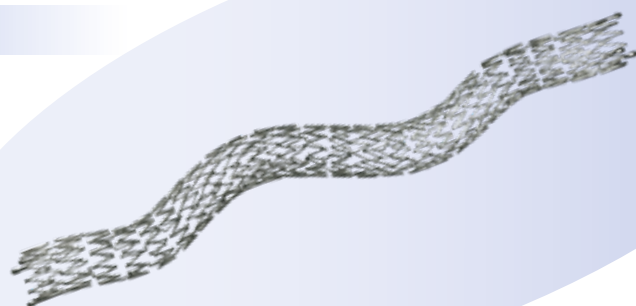

Conclusions

- More challenging population than typically enrolled in registry studies:
 - 24% CLI; longer, more complex lesions; >50% with DCB.
- 12-month freedom from CDTLR: 89%.
- No statistically significant difference in 12 month CDTLR for BioMimics 3D used with or without DCB.
- **Swirling flow is nature's way of protecting patency**
 - offering an alternative to drug elution to improve stenting outcomes.

The MIMICS Clinical Programme: An evolving database of the safety and effectiveness of the BioMimics 3D Vascular Stent System.

Gathering clinical evidence from a “real world” patient population from single de novo to complex, long and severely calcified lesions.

1750+
patients and
growing

| MIMICS FIH | MIMICS RCT | MIMICS 2 | MIMICS ^{3D} | MIMICS ^{3D} USA | MIMICS <i>et seq</i> |
|--|--|---|---|---|---|
| N = 10 1 site Germany | N = 50 8 sites Germany | N = 271 43 sites USA/Japan/Germany | N = 507 23 sites Pan European | N = c. 500 c. 40 sites USA | N = c. 400 Multiple sites Europe |
| <ul style="list-style-type: none"> • First in Human • FU - 1 year • Completed | <ul style="list-style-type: none"> • Randomised controlled trial • FU - 2 years • Completed | <ul style="list-style-type: none"> • IDE Registry • FU - 3 years • Completed | <ul style="list-style-type: none"> • Prospective Registry • FU - 3 years • 1 year complete | <ul style="list-style-type: none"> • Prospective Registry • FU - 3 years • Starting 2020 | <ul style="list-style-type: none"> • Physician initiated prospective and retrospective registries • Enrolment ongoing |

MIMICS RCT

A randomised study comparing safety and effectiveness of the BioMimics 3D Vascular Stent System to a straight stent control. Freedom from loss of primary patency through 2 years for BioMimics 3D Vascular Stent System was superior (P = 0.05) to straight control stents (72% vs 55%). There were no stent fractures at 2 years for patients treated with the BioMimics 3D stent.¹

MIMICS 2

A multicentre, international (USA, Japan and Germany) IDE study. At 3 years follow-up BioMimics 3D demonstrated continuing benefit with CDTLR showing comparable outcomes to DES/DCB. Core Lab X-ray imaging review confirmed 0% stent fracture in any MIMICS-2 subject treated with BioMimics. MIMICS-2 represents a more challenging patient population than in DES/DCB pivotal trials.^{2,3}

MIMICS^{3D}

A prospective observational registry evaluating the BioMimics 3D Vascular Stent System in a real-world clinical population with a dedicated subgroup analysis of device performance as a complementary treatment in procedures involving drug-coated balloons. MIMICS-3D enrolled 507 patients across 23 clinical sites in Europe.

MIMICS^{3D} USA

A prospective, multicentre observational study evaluating the safety, effectiveness and device performance of the BioMimics 3D Vascular Stent System within a real-world clinical population of patients undergoing femoropopliteal intervention. MIMICS-3D-USA will enrol more than 500 patients in 40 clinical sites across the United States.

1. Zeller T et al; Circ Cardiovasc Interv. 2016;9

2. Kenneth Rosenfield et al :N Engl J Med 2015;373:145-53. DOI: 10.1056/NEJMoa1406235

3. Michael D. Dake et al : Circ Cardiovasc Interv. 2011;4:495-504

The BioMimics 3D Vascular Stent System has CE Mark approval.

BioMimics 3D and Swirling Flow are registered trademarks of Veryan Medical Ltd. ©2020 Veryan Medical Ltd.

Indications, contraindications, warnings and Instructions for Use can be found in the product labelling supplied with each device.

All cited trademarks are the property of their respective owners.

For additional information please contact your local representative.

GERMANY
T +49 3222 999 0027
E veryanmedical@healthlinkeurope.com
W veryanmed.com

ALL OTHER EUROPEAN COUNTRIES
T +31 (0)73 303 5510
E veryanmedical@healthlinkeurope.com
W veryanmed.com